
ARROW EZ-IO[®]
INTRAOSSEOUS VASCULAR ACCESS

EZ-IO[®] INTRAOSSEOUS VASCULAR ACCESS NEEDLES

Instructions for Use

English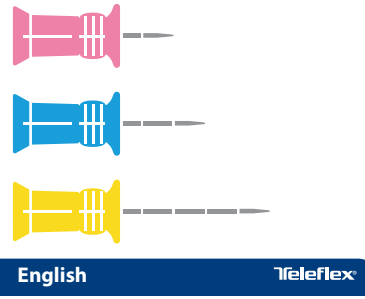

INDICATIONS FOR USE:
For introsseous access anytime in which vascular access is difficult to obtain in emergent, urgent or medically necessary cases.

ADULTS:
Proximal humerus
Proximal tibia
Distal tibia

PEDIATRICS:
Proximal humerus
Proximal tibia
Distal tibia
Distal femur

CONTRAINDICATIONS FOR USE:
Fracture in target bone
Previous, significant orthopedic procedure at the site, prosthetic limb or joint.
IO catheter use in past 48 hours of target bone
Infection at the area of insertion.
Torsion (twist (onset clarity) and/or absence of adequate anatomical landmarks.


WARNINGS AND PRECAUTIONS FOR EZ-IO[®] INTRAOSSEOUS VASCULAR ACCESS SYSTEM:
Use aseptic technique.
CAUTION: Check skin, adipose and muscle thickness before insertion.
CAUTION: Extra care should be taken during insertion and site monitoring when used in patients with bone diseases that increase the likelihood of fracture, extravasation and dislodgement.
CAUTION: Do not recap Needle Sets or reconnect separated components. The hub and sharp display protrusions. Re-use of contents may cause cross-contamination, leading to patient risk and complications.
CAUTION: Before administering vesicant, toxic, or highly-concentrated drugs, check the IO Catheter again for placement and patency.
CAUTION: Use caution with chemotherapeutic agents.
CAUTION: Monitor insertion site frequently for extravasation.
CAUTION: Stylet and Catheter are NOT heat compatible.
CAUTION: Do not leave the Catheter inserted for longer than 24 hours.
CAUTION: Needle Sets are single use only, serious medical consequences (e.g. Life threatening infection) and reduced performance (e.g. Buried needles) may occur if compliance to this warning is not followed. For a complete listing of these serious medical and performance consequences, please contact Teleflex.
STORAGE: Store in a cool dry place.

ARROW EZ-IO[®]

EZ-IO[®] NEEDLE SETS, DESCRIPTION

- Comprised of Catheter with Luer-lock connection, Stylet, Safety Cap.
- 15 gauge, 304 stainless steel in 15 mm, 25 mm and 45 mm lengths.
- Sterile, non-pyrogenic, in protective packaging.
- Intended for use with EZ-IO[®] Power Driver.

EZ-IO[®] Power Driver and Needle Sets Description

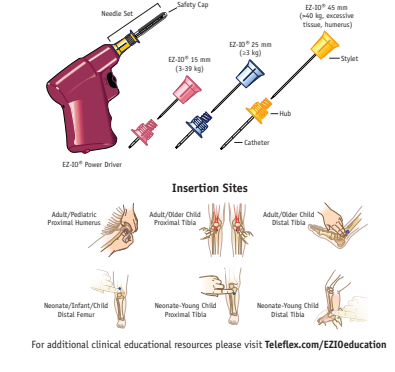Insertion Sites

For additional clinical education resources please visit Teleflex.com/EZIOeducation

ARROW EZ-IO[®]

1. Clean insertion site per institutional protocol/policy. Prepare supplies.
 - Prime EZ-Connect[™]
 - Ultrasound
 - Prime set and purge air.
 - Use EZ-Connect[™] packaging.
2. Attach Needle Set to EZ-IO[®] Power Driver and remove Safety Cap from Catheter. IMPORTANT: Only handle Needle Set by the plastic hub.
3. Push Needle Set through skin until tip touches bone.
4. Remove Safety Cap from Needle Set.
5. 5 mm of the Catheter (at least one black line) must be visible outside the skin. IMPORTANT: The most accurate determination of correct needle selection is use of depth markings. Black depth marks on each catheter function as depth measuring guides to determine soft tissue depth, avoiding bone. (See above).
6. General recommendations for needle set selection include the following (depth markings are not used for needle selection):
 - 15 mm: Neonates and small infants proximal and distal tibia.
 - 25 mm: Neonates and small infants proximal and distal tibia.

ARROW EZ-IO[®]

7. Advance trigger and apply moderate steady pressure. IMPORTANT: Do NOT USE EXCESSIVE FORCE. Use moderate steady downward pressure and STOP Needle Set rotation to prevent the bone.
8. Note EZ-IO[®] Power Driver and Needle Set will penetrate the bone, operator may be applying too much downward pressure to prevent bone.
9. Note: In the event of a Drive Failure, disconnect the Power Driver, keep the Needle Set Hub and any contents into the medullary space while waiting.
10. Advance Needle Set and release trigger. IMPORTANT: Release trigger when sudden "give" or "pop" is felt, indicating entry into medullary space.
11. Pull back Needle Set approximately 1-2 cm after entry into medullary space. In Adults: Advance Needle Set should be advanced 2 cm or until Hub is flush against the skin. In Pediatric: Advance Needle Set should be advanced 2 cm or until Hub is flush against the skin.
12. Stabilize Needle Set Hub, disconnect Driver, and remove Stylet.
13. Place Stylet into Needle/SET[®] for sharp containment.
NOTE: After the Needle/SET[®] is in place, the operator should immediately following use of a Needle and while still holding it with one hand away from the sharp end, firmly insert the sharp point just under the skin.
NOTE: Do not use the sharp point to pierce the skin. Do not use the sharp point to pierce the skin.
FREE HAND AWAY FROM THE SHARP END DURING INSERTION. ALWAYS USE ONE-HANDED TECHNIQUE WHEN INSERTING SHARP. Do not use a sharp object to pierce the skin.
NEEDLE/SET[®] whether or not it has been used.
14. Obtain samples for lab analysis, if needed.
NOTE: Only attach a Syringe directly to the EZ-IO[®] Catheter Hub when drawing blood for laboratory analysis (anterior collection) or removal.
15. Use of the EZ-Stabilizer[™] is strongly recommended for all EZ-IO[®] Insertions.
 - a. Place Stabilizer over Catheter Hub.
 - b. Attach a primed EZ-Connect[™] extension set to the Hub, firmly secure by twisting clockwise.
 - c. Do not use any instrument to tighten connections.
 - d. Note: Do not use a sharp instrument to pierce the bone.
 - e. Note: Do not use a sharp instrument to pierce the bone.
 - f. Note: Do not use a sharp instrument to pierce the bone.
 - g. Note: Do not use a sharp instrument to pierce the bone.
16. Remove EZ-Connect[™] and use a syringe to aspirate the catheter.
17. Document date/time of insertion and apply EZ-IO[®] wristband.
CAUTION: Monitor insertion site frequently for extravasation.
CAUTION: Do not leave the Catheter inserted for longer than 24 hours.

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©2014 Teleflex Incorporated. Arrow[®], EZ-IO Intraosseous Vascular System[™], EZ-Stabilizer[™] or EZ-Connect[™] are trademarks of Teleflex Inc.