ARROW®

Sheath Adapter Product with Cath-Gard® Catheter Contamination Shield for use with 4 Fr. - 7 Fr. Catheters

Rx only.

Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged.

Warning: Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.

Do not alter the sheath adapter, Cath-Gard® or any other kit/set component during insertion, use, or removal.

Procedure must be performed by trained personnel well versed in anatomical landmarks, safe techniques, and potential complications.

Indications for Use:

The Arrow Sheath Adapter is utilized in conjunction with one-piece Percutaneous Sheath Introducer to permit venous access and catheter introduction into the central circulation.

Contraindications:

None known.

Warnings and Precautions:*

1. **Warning:** Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.
2. **Warning:** The practitioner must be aware of potential air embolism associated with leaving open needles, sheaths or catheters at venous puncture sites or as a consequence of inadvertent disconnection. To reduce the possibility of disconnection, use bayonet lock connections only. Follow hospital protocol for sheath and adapter maintenance to guard against air embolism.
3. **Warning:** The connection between sheath adapter and sheath must be tightly secured and examined routinely to minimize the risk of disconnection and possible air embolism, blood loss, or exsanguination.^{3,4}
4. **Warning:** Sheath adapter should be removed immediately following catheter removal and replaced with Arrow obturator or obturator cap. During catheter introduction or exchange, the adapter must be occluded at all times to minimize the risk of air embolism or hemorrhage. If catheter introduction is delayed or catheter is removed, temporarily cover adapter opening with sterile-gloved finger until either catheter is inserted, sheath adapter is completely closed or an obturator or obturator cap is attached.
5. **Warning:** Due to the risk of exposure to Human Immunodeficiency Virus (HIV) or other blood borne pathogens, health care professionals should always use universal blood and body fluid precautions in the care of all patients.
6. **Warning:** Percutaneous sheath introducers and adapters should be maintained in closely monitored environments (e.g. critical care unit, operating room, or recovery room), and not in a general nursing unit.⁵

7. **Warning:** Do not apply excessive force in removing catheter or sheath. If withdrawal is difficult, then a chest x-ray should be obtained and further consultation requested.
8. **Precaution:** Sheath adapter with Cath-Gard® should be used with one-piece 8.5 Fr. and 9 Fr. percutaneous sheath introducers only.
9. **Precaution:** Sheath adapter allows passage of 4 Fr. to 7 Fr. catheters only.
10. **Precaution:** If excessive force is needed in securing adapter on sheath, do not use that adapter.
11. **Precaution:** Perform routine insertion site care and maintenance to cap of adapter especially before inserting catheter. Make sure cap is cleaned per hospital protocol before reinserting catheter.

A Suggested Procedure:

Use sterile technique.

1. Place patient in desired position.
2. Prep and drape puncture site as required.
3. Insert percutaneous sheath introducer into the patient according to the manufacturer's instructions.
4. Attach sheath adapter to hub of one-piece percutaneous sheath introducer and lock it in place via bayonet lock mechanism (refer to Fig. 1). Orient the slot in sheath adapter with the locking pin on sheath hub and slide the adapter forward over sheath hub. Twist to lock. **Warning:** The connection between sheath adapter and sheath must be tightly secured and examined routinely to minimize the risk of disconnection and possible air embolism, blood loss, or exsanguination. **Precaution:** Sheath adapter should be used with one-piece 8.5 Fr. and 9 Fr. percutaneous sheath introducers only. If excessive force is needed in securing adapter on sheath, do not use that adapter.

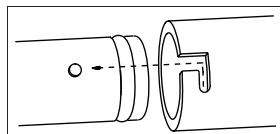
Fig. 1

5. Insert tip of desired catheter through Tuohy-Borst (T-B) cap end of Cath-Gard®. Advance catheter through tubing and soft hub at distal end of Cath-Gard® (refer to Fig. 2). **Precaution:** Sheath adapter allows passage of 4 Fr. to 7 Fr. catheters only.

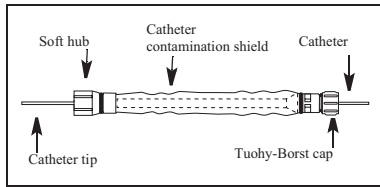
Fig. 2

6. Feed catheter through sheath adapter and sheath into vessel and advance catheter to desired position. **Warning: Sheath adapter must be occluded at all times to minimize the risk of air embolism or hemorrhage. If catheter introduction is delayed or catheter is removed, temporarily cover adapter opening with sterile-gloved finger until either catheter is inserted, sheath adapter is completely closed or an obturator or obturator cap is attached.**
7. Twist sheath adapter cap clockwise to lock catheter in position. Tighten cap to the point that a slight resistance is encountered when catheter is gently tugged to ensure tight seal around it.
8. Slide entire catheter contamination shield to proximal end of catheter so that soft hub of Cath-Gard® is approximately 5 inches (12.7 cm) from sheath adapter (refer to Fig. 3).

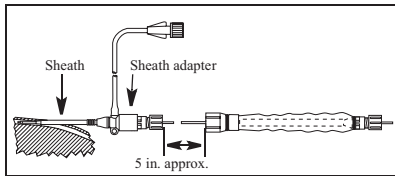
Fig. 3

9. Secure T-B cap end of Cath-Gard® in place by turning the cap clockwise until slight resistance is encountered. Disengage soft hub of Cath-Gard® from inner feed tube by pulling forward and advance it forward towards sheath adapter.
10. Secure soft hub over adapter cap and make sure connection is secure.
11. Grasp catheter through front portion of Cath-Gard® catheter contamination shield and hold in place while repositioning T-B cap end as desired (refer to Fig. 4). Tighten the cap of T-B adapter also to minimize the risk of migration of Cath-Gard® catheter contamination shield. Apply sterile sealing tape to T-B cap and catheter junction to establish and maintain sterile field within Cath-Gard®.

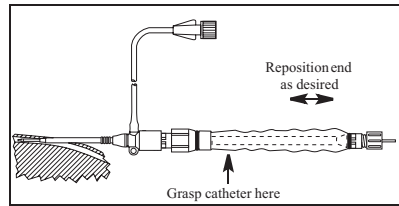
Fig. 4

12. If catheter introduction is delayed or catheter is removed, temporarily cover adapter opening with sterile-gloved finger until either catheter is inserted or sheath adapter is completely removed. **Warning: Sheath adapter must be occluded at all times to minimize the risk of air embolism or hemorrhage.**
13. Remove adapter from percutaneous sheath introducer if it is no longer needed to introduce catheter and place an Arrow obturator cap or obturator via bayonet lock mechanism on the sheath hub to establish hemostasis.⁵
14. Dress sheath insertion site per hospital protocol. **Precaution: Maintain the insertion site with regular, meticulous redressing using aseptic technique.**

Reference:

1. ECRI. Air embolism and exsanguination from separation of two-piece side port/hemostasis valve cardiac catheter introducers. Health Devices Alerts. January 1995;24:36-38.
2. ECRI. Catheter Introducers – hemostasis valve. Health Devices Alerts. February 3, 1995-A05:1.
3. Hartung EJ, Ender J, Sgouropoulou S, Bier R, Engelhardt W, Engemann R. Severe air embolism caused by a pulmonary artery introducer sheath. Anesthesiology. 1994; 80:1402. Letter.
4. Kondo K, O'Reilly LP, Chiota J. Air embolism associated with an introducer for pulmonary arterial catheters. Anesth Analg. 1984; 63:871-872.
5. Mihm FG, Rosenthal MH. Pulmonary artery catheterization. In: Benumof JL, ed. Clinical Procedures in Anesthesia and Intensive Care. Philadelphia, PA: JB Lippincott; 1992:419. Tocino IM, Watanabe A. Impending catheter perforation of superior vena cava: radiographic recognition. *Am J Roentgenology*. March 1986;146:487-490.

Arrow International, Inc. recommends that the user be acquainted with the reference literature.

*If you have any questions or would like additional reference information, please contact Arrow International, Inc.

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Teleflex, the Teleflex logo, Cath-Gard are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.
© 2016 Teleflex Incorporated. All rights reserved.
Arrow International, Inc. rekommenderar att användaren studerar referenslitteraturen.
*Om du har frågor eller önskar ytterligare referensinformation, kontakta Arrow International, Inc.