Arrow-Johans™ Right Atrial Electrocardiography (RAECG) Adapter

Rx only.

Indications for Use:

The RAECG Adapter permits verification of catheter tip position using electrocardiography.

Contraindications:

None known.

General Warnings and Precautions

Warnings:

1. Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.
2. Read all package insert warnings, precautions and instructions prior to use. Failure to do so may result in severe patient injury or death.
3. Strict electrical isolation of monitoring equipment and good grounding techniques are mandatory when using a saline-filled catheter as a transvenous ECG electrode. All equipment should comply with American Heart Association recommendation of being incapable of delivering more than 10 microamp leakage current to the patient. An electrocautery protection filter should be inserted between ECG cable and ECG machine. Rubber cap must be over stainless steel male RAECG button when adapter is in-line but not in use.
4. Clinicians must be aware of complications/undesirable side-effects associated with right atrial and inadvertent right ventricular catheterization whenever central venous catheter being introduced is intentionally passed beyond junction of superior vena cava and right atrium.
5. Any premature heart beat might indicate catheter touching heart; withdraw catheter and check position.

Precautions:

1. Do not alter the adapter during use.
2. Procedure must be performed by trained personnel well versed in anatomical landmarks, safe technique and potential complications.
3. Use standard precautions and follow institutional policies for all procedures including safe disposal of devices.

A Suggested Procedure: Use sterile technique.

1. Aseptically place catheter intravenously.
2. Sterile Operator – Pass distal or female end of a sterile length of IV tubing to non-sterile assistant.

3. Non-sterile Assistant – Luer-Lock adapter to IV tubing and flush system with an ionic salt solution, e.g. 0.9% saline solution, 4% sodium chloride or 8.4% sodium bicarbonate solution. Meticulously remove all air bubbles and blood clots.
4. Sterile Operator – Luer-Lock male or proximal end of sterile IV tubing to catheter.
5. Non-sterile Assistant – Flush entire assembly.
6. Sterile Operator – Remove rubber protective cap from stainless steel male button on adapter.
7. Non-sterile Assistant – For ECG machines without lead selectors, connect negative terminal lead of ECG to adapter, the positive terminal to left leg, and the ground terminal lead to left arm. For ECG machines with lead selectors, choose lead II on ECG machine and place right arm lead to adapter.
8. Sterile Operator – Advance or withdraw saline filled catheter until size of the P wave of ECG trace just begins to progressively get larger. At this point, catheter tip should be at cavoatrial junction. If tip is advanced further into right atrium, P wave will either be as large as R wave or become biphasic. If catheter enters right ventricle, P wave will disappear and a wider QRS will appear.
9. Continue with procedure.

Trouble Shooting Tips:

- Salt bridge solution – The solution in the catheter must be highly ionic to conduct the electrical impulse. Do not use dextrose in water since it is non-ionic.
- Flush fluids – Discontinue all flush fluids to minimize the risk of electrical deflection.
- Grounding – Ensure good skin contact with ECG electrodes to optimize ECG tracing.
- Internal catheter movement – Movement of chest during respiration or movement of heart during contraction can cause catheter to move and produce an undulating trace. Repositioning chest leads may alleviate problem.

For reference literature concerning patient assessment, clinician education, insertion technique, and potential complications associated with this procedure, consult standard textbooks, medical literature, and Arrow International LLC website: www.teleflex.com

A pdf copy of this IFU is located at www.teleflex.com/IFU

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority. The contacts of national competent authorities (Vigilance Contact Points) and further information can be found on the following European Commission website: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

en

Symbol Glossary: Symbols are in compliance with ISO 15223-1

								
Caution	Medical device	Consult instructions for use	Do not reuse	Do not resterilize	Sterilized by ethylene oxide	Single sterile barrier system	Keep away from sunlight	Keep dry
								
Do not use if package is damaged	Not made with natural rubber latex	Catalogue number	Lot number	Use by	Manufacturer	Date of manufacture	Importer	

Teleflex, the Teleflex logo, Arrow, the Arrow logo and Arrow-Johans are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. © 2020 Teleflex Incorporated. All rights reserved.

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