

USER-GARD™

NEEDLELESS INJECTION HUB

INSTRUCTIONS FOR USE:

1. Attach luer end of User-Gard™ hub to syringe.
2. Prepare injection site with alcohol or betadine per standard hospital protocol.
3. Remove red dust cap.
4. Press User-Gard™ hub onto injection site and twist to lock on pin (see fig. 1).

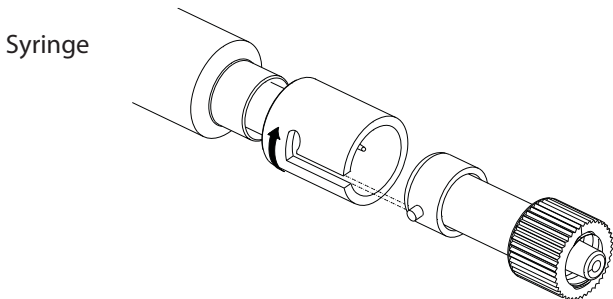


FIG. 1

5. Inject or withdraw fluid as required.
6. Disengage User-Gard™ hub from injection site and discard. **Caution: To prevent possible air embolism, do not leave User-Gard™ connected to injection site.** Single use only.