Inspected Dimensions:
Folded Length: 7-1/2"
Folded Width: 5-1/2"

ARROW®

Arrow Raulerson Spring-Wire Introduction Syringe

Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged. **Warning:** Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.

Do not alter the syringe.

Syringe must be used by trained personnel well versed in anatomical landmarks, safe technique and potential complications.

Indications for Use:

The Arrow Raulerson Syringe is indicated for use with spring-wire guides ranging from .025" (.64 mm) to .038" (.96 mm) and spring-wire guide introduction needles sized to place these wires. The syringe allows for vessel location and passage of the guide wire through the syringe obviating the need for separating the introducer needle from the syringe.

Contraindications:

None known.

Warnings and Precautions:*

- Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.**
- Warning: Practitioners must be aware of complications associated with vessel access and spring-wire guide placement including vessel wall perforation, air embolism, guide wire embolism, bacteremia, septicemia, inadvertent arterial puncture, guide wire malposition, hematoma, dysrhythmias, and nerve damage.**
- Warning: The practitioner must be aware of potential air embolism problems associated with leaving open needles in venous puncture sites.**
- Warning: Care should be exercised in passing spring-wire guides. Use of excessive length of guide wire into the right heart can cause dysrhythmias, right bundle branch block,¹ and vessel wall, atrial or ventricular perforation.**

- Warning: Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.**
- Precaution: The color of the blood aspirated is not always a reliable indicator of venous access.² For central venous catheter insertion use the Arrow Raulerson Syringe in conjunction with the transduction probe to verify central venous placement via a wave form obtained by a calibrated pressure transducer (refer to Fig. 1).**

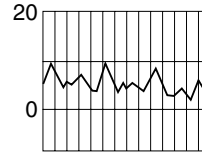
Fig. 1

If hemodynamic monitoring equipment is not available to permit transducing a central venous wave form, disconnect the syringe from the needle and check for pulsatile flow. Pulsatile flow is usually an indicator of inadvertent arterial puncture.

A Suggested Procedure:

Use sterile technique.

- Precaution: Place patient in slight Trendelenburg position as tolerated to reduce the risk of air embolism. If femoral approach is used, place patient in supine position.**
- Prep area of anticipated puncture site.
- Drape puncture site as required.
- Perform skin wheal utilizing appropriate anesthetic agent and needle.
- Insert introducer needle attached to Arrow Raulerson Syringe into vessel and aspirate. Vessel may be prelocated with a 22 Ga. locator needle. **Precaution: The color of the blood aspirated is not always a reliable indicator of venous access.²** For pressure wave form transduction, a separately packaged transduction probe is available (Product

No. AR-14703). Contact Arrow International for additional details or reference information.





6. Straighten spring-wire guide "J". Advance spring-wire guide through syringe into vessel to desired depth. **Warning: Do not withdraw spring-wire guide against needle bevel to minimize the risk of possible severing or damaging of spring-wire guide. Precaution: Do not reinfuse blood to minimize the risk of blood leakage from the rear of syringe. Precaution: Do not aspirate with spring-wire guide in place or air will enter syringe.**
7. Hold spring-wire guide in place and remove introducer needle and Raulerson Syringe. **Precaution: Maintain firm grip on spring-wire guide at all times.**
8. Proceed with procedure per hospital protocol or manufacturer's instructions.

References:

1. Eissa NT, Kvetan V. Guide wire as a cause of complete heart block in patients with preexisting left bundle branch block. *Anesthesiology*. 1990;73:772-774.
2. Jobes DR, Schwartz AJ, Greenhow DE, Stephenson LW, Ellison N. Safer jugular vein cannulation: recognition of arterial punctures and preferential use of the external jugular route. *Anesthesiology*. 1983;59:353-355.

Arrow International, Inc. recommends that the user be acquainted with the reference literature.

*If you have any questions or would like additional reference information, please contact Arrow International, Inc.

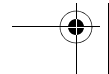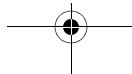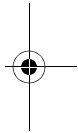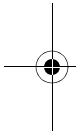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Arrow International, Inc. rekommenderar att användaren studerar referenslitteraturen.
* Om du har frågor eller önskar ytterligare referensinformation, kontakta Arrow International, Inc.