## Peripherally Inserted Central Catheter (PICC) Patient Information Booklet

EN

Patient's Name: \_\_\_\_\_

Doctor's Name: \_\_\_\_\_

Telephone Number: \_\_\_\_\_

Nurse's Name: \_\_\_\_\_

Telephone Number: \_\_\_\_\_

Home Care Agency: \_\_\_\_\_

Telephone Number: \_\_\_\_\_

Contact Name: \_\_\_\_\_

Catheter Information: \_\_\_\_\_

Catheter Coating: Coated: \_\_\_\_\_ Non-coated: \_\_\_\_\_

Date Inserted: \_\_\_\_\_

Inserted by: \_\_\_\_\_

Lot Number: \_\_\_\_\_

Catheter Length: \_\_\_\_\_

### Introduction:

Your doctor has requested that a PICC be inserted for your medical treatment. This booklet has been given to you to provide information about the Arrow® PICC. Use it in addition to the directions you have received from your nurse or doctor.

PICC stands for a "Peripherally Inserted Central Catheter". It is a small tube that is placed into a blood vessel in one of your arms and the tip is advanced to a position that is close to your heart (refer to Figure 1). Medications or nutritional fluids can be given through the catheter for an illness that requires treatment over a period of time. A PICC is an alternative to having repeated needle sticks or surgery to place another kind of intravenous (IV) device.

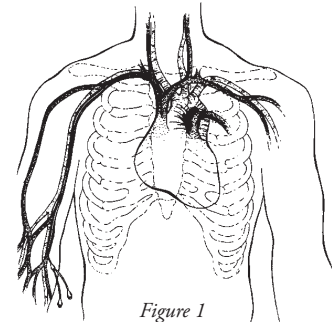
Figure 1

The PICC consists of a long, hollow tube (lumen) with one (or more) holes at the tip that lies within the body and a hub at the other end that is outside of the body on the arm surface (refer to Figure 2). Medications or fluids can be given through the hub continuously or injected periodically.

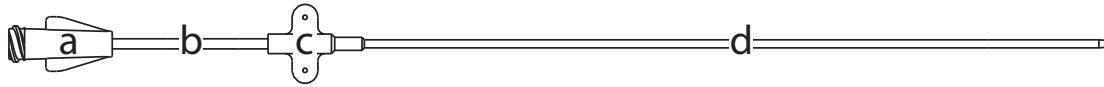
Figure 2: a) Infusion Hub b) Extension Line c) Anchoring Hub d) Catheter Tube (Lumen)

### Catheter Insertion:

The catheter is inserted by a doctor or nurse into a blood vessel on the inside of your arm near where the elbow bends. Measurements are taken to estimate the amount of catheter that needs to be inserted. An x-ray may be taken to provide a picture of where the catheter tip lies within your body. The doctor or nurse will use sterile technique to insert the catheter. This means that precautions will be taken to make sure that germs are not inserted with the catheter. Similar care should be taken with all further catheter-related procedures.

### Catheter Care and Maintenance:

#### Dressings:

The insertion site and external portion of the PICC should be covered with a protective dressing. The dressing will help to hold the catheter in place and keep the area clean and protected from germs.

Because the catheter is small and pliable, the catheter body can be anchored in place by using stitches, sterile tape strips or a special anchoring device. The special anchoring device consists of a catheter clamp and a crescent-shaped adhesive strip. The strip is placed on the arm and the catheter clamp that has been applied to the catheter is snapped into place (refer to Figures 3, 4, and 5). Another dressing, either gauze or a transparent sheet, is placed over the site as a final protective measure.

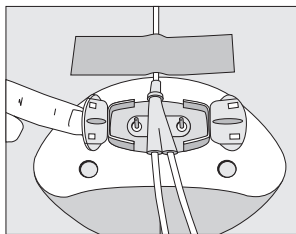
Figure 3

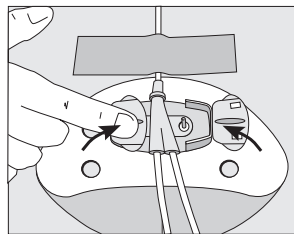
Figure 4

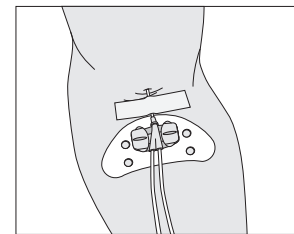
Figure 5

If you or a family member have been trained to redress your catheter, follow the directions given by your doctor or nurse. Be sure to wash your hands prior to starting the procedure and use sterile technique as you have been instructed. As a safeguard, do not use sharp instruments such as scissors during the dressing change to avoid accidentally cutting your catheter. Inspect the insertion site regularly for any redness or drainage, and watch the catheter to make sure it does not move in or out further than when it was inserted. If the dressing becomes loose, make sure that it is changed.

#### Flushing:

To keep your PICC open to flow, it may be flushed and filled with a weak sterile salt solution also known as sterile normal saline or an anti-clotting solution. The solution keeps blood from backing up into the lumen and forming a clot. If you or a family member are caring for your catheter, follow the directions given by your doctor or nurse. They will tell you what solution should be used as well as the amount that should be given and how often the catheter should be flushed.

#### Complications:

Although you should not expect a problem to occur that is related to your PICC, it is important that you become familiar with the signs and symptoms that might signal such a problem. See the following table.

Signs and Symptoms	Possible Cause	Directions	Prevention
<p>If your PICC has the special infection protection coating of chlorhexidine, you need to be aware of the possible allergic reaction(s) associated with the use of chlorhexidine</p> <p>Redness at insertion site, hives, itching, anxiety, chest discomfort or tightness, cough, difficulty breathing, difficulty swallowing, abnormal (high-pitched) breathing sounds, dizziness or light-headedness, flushing or redness of the face, nausea or vomiting, palpitations, swelling of the face, eyes, or tongue, unconsciousness, wheezing</p>	<p>Allergic reaction to <b>chlorhexidine</b></p> <p>Most severe allergic reactions occur within seconds or minutes after catheter insertion when you will be in the presence of the nurse or doctor, however, some reactions can occur after several hours</p> <p>Anaphylaxis is the severest reaction which can occur</p> <p>It occurs suddenly and within minutes of catheter insertion and immediate medical attention is needed</p>	<p>Tell the nurse or doctor about any of these symptoms while the catheter is being inserted or as they occur</p>	<p>Inform the doctor or nurse of any allergies associated with chlorhexidine before having the catheter inserted</p>
<p>Drainage, redness, pain, swelling around insertion site, fever and chills</p>	<p>Infection</p>	<p>Call doctor or nurse</p>	<p>Use sterile technique Keep sterile dressing over site Wash hands prior to procedures</p>
<p>Arm or shoulder swelling, swishing in ear on same side of body where catheter is located while medication given</p>	<p>Catheter position change</p>	<p>Call doctor or nurse Do not inject any solutions into catheter until talking with doctor or nurse</p>	<p>Inject flushing/locking solution slowly</p>
<p>Inability to inject</p>	<p>Catheter clotted or kinked</p>	<p>Call doctor or nurse</p>	<p>Completely fill catheter lumen with locking solution between treatments Flush catheter well before and after medications</p>
<p>Leaking from external catheter</p>	<p>Break in catheter material, hub separation</p>	<p>Call doctor or nurse Fold catheter together below leaking area and tape securely</p>	<p>Do not use alcohol or acetone (as in nail polish or tape remover) on catheter Do not pull on catheter</p>
<p>Pain on injection</p>	<p>Inflammation of vein</p>	<p>Call doctor or nurse</p>	<p>Medications should be given slowly</p>

**Questions:**

The following questions cover topics concerning your daily routine while your PICC is in place. You may want to discuss them with your doctor or nurse.

- Are there any limits to my activities?

\_\_\_\_\_

- May I bathe/shower and swim while my catheter is in place?

\_\_\_\_\_

- How often should my dressing be changed?

\_\_\_\_\_

- How often should my catheter be flushed and what solution should be used?

\_\_\_\_\_

- What fluids and medications will I be receiving through my catheter?

\_\_\_\_\_

- Are there any special instructions concerning my IV medications?

\_\_\_\_\_

- Who should I call if I have questions about my catheter?

\_\_\_\_\_

### Catheter Care Schedule

Date	Dressing Change	Flushing Procedure	Tubing Change	Cap Change

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