Inspected Dimensions:
Folded Length: 7-1/2" (19 cm)
Folded Width: 5-1/2" (14 cm)

ARROW®

EN

SharpsAway II™ Locking Disposal Cup

Indications for Use:

The Arrow® SharpsAway II™ Locking Disposal Cup is intended as a single use needle protection device, which covers the end of a needle after use to help prevent needle sticks.

Contraindications:

None known.

Warnings and Precautions:

- Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.**
- Warning: Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.**
- Precaution: Properly dispose of sharps in sharps container in accordance with state/federal OSHA standards for blood borne pathogens and/or institutional policy.**

A Suggested Procedure: Use sterile technique.

- The SharpsAway II™ Locking Disposal Cup is used for disposal of needles (15 Ga. - 30 Ga.).
 - Using one-handed technique, firmly push needles into disposal cup holes (refer to Fig. 1).
 - Once placed into disposal cup, needles will be automatically secured in place so that they cannot be reused.
 - Discard entire cup, at completion of procedure, into an approved sharps container.
- Precaution: Do not attempt to remove needles that have been placed into cup. These needles are secured in place. Damage may occur to needles if they are forced out of disposal cup.**

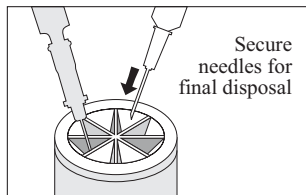
Fig. 1

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