SharpsAway® II Locking Disposal Cup

Rx only.

Indications for Use:

The Arrow® SharpsAway II Locking Disposal Cup is intended as a single use needle protection device, which covers the end of a needle after use to help prevent needle sticks.

Contraindications:

None known.

⚠️ General Warnings and Precautions

Warnings:

- Sterile, Single use: Do not reuse, reprocess or resterilize.** Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.
- Read all package insert warnings, precautions and instructions prior to use. Failure to do so may result in severe patient injury or death.

Precautions:

- Do not alter component.
- Use standard precautions and follow institutional policies for all procedures including safe disposal of devices.

A Suggested Procedure: Use Sterile Technique.

The SharpsAway II Locking Disposal Cup is used for disposal of needles (15 Ga. - 30 Ga.).

- Use one-handed technique, firmly push needles into disposal cup holes (refer to Figure 1).

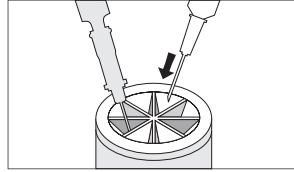
Figure 1

- Once placed into disposal cup, needles will be automatically secured in place so that they cannot be reused.
- Discard entire cup, at completion of procedure, into an approved sharps container.

⚠️ **Precaution:** Do not attempt to remove needles that have been placed into cup. These needles are secured in place. Damage may occur to needles if they are forced out of disposal cup.

For reference literature concerning patient assessment, clinician education, insertion technique, and potential complications associated with this procedure, consult standard textbooks, medical literature, and Arrow International LLC website: www.teleflex.com

A pdf copy of this IFU is located at www.teleflex.com/IFU

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority. The contacts of national competent authorities (Vigilance Contact Points) and further information can be found on the following European Commission website: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

Symbol Glossary: Symbols are in compliance with ISO 15223-1.

Some symbols may not apply to this product. Refer to product labeling for symbols that apply specifically to this product.

Caution	Medical device	Consult instructions for use	Do not reuse	Do not resterilize	Sterilized by ethylene oxide	Single sterile barrier system	Keep away from sunlight	Keep dry
Do not use if package is damaged	Not made with natural rubber latex	Catalogue number	Lot number	Use by	Manufacturer	Date of manufacture	Importer	

Arrow, the Arrow logo, SharpsAway, Teleflex, and the Teleflex logo are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. © 2021 Teleflex Incorporated. All rights reserved.

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