ARROW®

SharpsAway II™ Locking Disposal Cup

Indications for Use:

The Arrow® SharpsAway II™ Locking Disposal Cup is intended as a single use needle protection device, which covers the end of a needle after use to help prevent needle sticks.

Contraindications:

None known.

Warnings and Precautions:

1. **Warning:** Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.
2. **Precaution:** Properly dispose of sharps in sharps container in accordance with state/federal OSHA standards for blood borne pathogens and/or institutional policy.

A Suggested Procedure: Use sterile technique.

1. The SharpsAway II™ Locking Disposal Cup is used for disposal of needles (15 Ga. - 30 Ga.).
 - Using one-handed technique, firmly push needles into disposal cup holes (refer to Fig. 1).
 - Once placed into disposal cup, needles will be automatically secured in place so that they cannot be reused.
 - Discard entire cup, at completion of procedure, into an approved sharps container.
2. **Precaution:** Do not attempt to remove needles that have been placed into cup. These needles are secured in place. Damage may occur to needles if they are forced out of disposal cup.

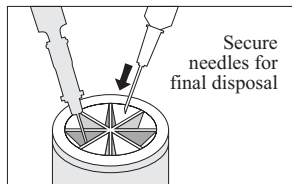
Fig. 1

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