


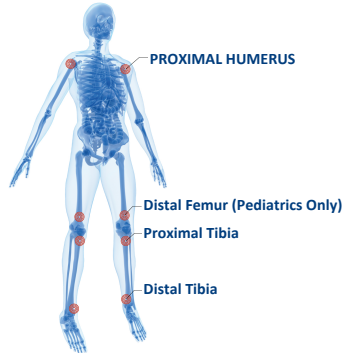


IDENTIFY

Choose insertion site




LANDMARK

Position extremity, landmark, and clean site

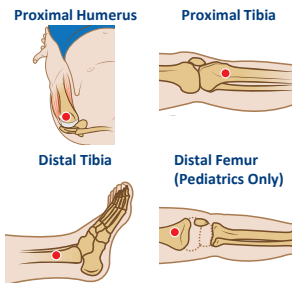


ACCESS

Attach needle set and remove orange tab on EZ-IO™ Power Driver (Single-Use)

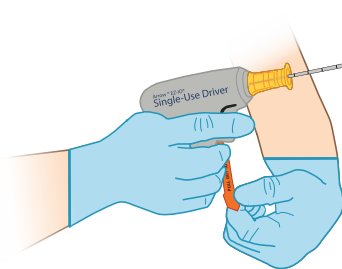


Insert and touch bone

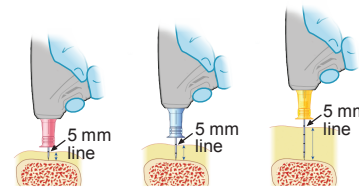


The 5 mm line, closest to the hub, must be visible above skin for needle set length confirmation

Engage EZ-IO™ Power Driver (Single-Use)

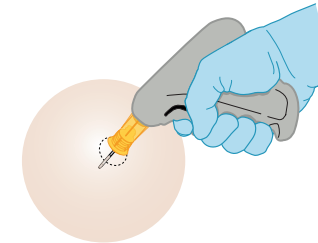


ACCESS (Continued)

Remove stylet and secure in SharpsAway™ II Locking Disposal Cup

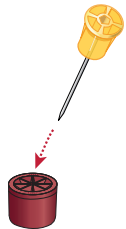


Apply EZ-Stabilizer™ Dressing

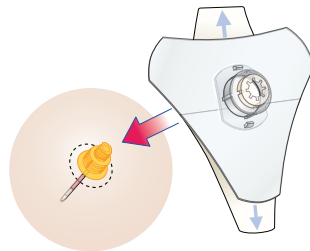


DELIVER

Attach primed EZ-Connect™ Extension Set and rapid flush with saline

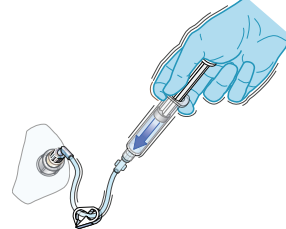

Administer fluids and medication

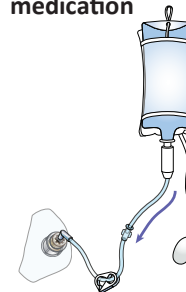

RESOURCES

24-Hour Clinical Hotline
+1-888-413-3104
clinical.affairs@teleflex.com

Customer Service
+1-866-479-8500
cs@teleflex.com

ArrowEZIO.com
LBL069430 R02 (2024-01)

Images are representative. Refer to Instructions for Use for complete instructions, warnings, indications, contraindications, precautions, and potential complications.

