Inspected Dimensions:
Folded width: 5-1/2" (14 cm)
Folded length: 7-1/2" (19 cm)

Peel-Away Sheath over Needle Assembly

Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged. **Warning: Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.**

Do not alter the assembly during insertion, use, or removal.

Procedure must be performed by trained personnel well versed in anatomical landmarks, safe technique, and potential complications.

Indications for Use:

The Peel-Away Sheath over Needle Assembly permits venous access for catheter introduction.

Contraindications:

None known.

Warnings and Precautions:*

1. **Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.**
2. **Warning: Do not withdraw peel-away introducer sheath against needle.**
3. **Warning: Do not leave the peel-away introducer sheath in place as an IV access site.**
4. **Warning: Due to the risk of exposure to HIV (Human Immunodeficiency Virus) or other blood borne pathogens, health care workers should routinely use universal blood and body-fluid precautions in the care of all patients.**

A Suggested Procedure: Use sterile technique.

1. After the pre-insertion measurements and patient preparations have been completed, perform venipuncture using peel-away introducer sheath over needle assembly.
2. Advance peel-away introducer sheath into vessel and remove introducer needle. **Warning: Do not reinsert introducer needle into peel-away sheath.** Observe for pulsatile flow. Pulsatile flow is usually an indicator of inadvertent arterial puncture.
3. Insert distal end of flushed catheter into peel-away sheath, and slowly advance the catheter.
4. Prior to advancing catheter to final indwelling position, carefully withdraw peel-away introducer sheath until free from venipuncture site.
5. Grasp tabs of peel-away introducer sheath and pull apart, away from indwelling catheter, until introducer sheath splits down entire length. Dispose of introducer sheath segments.
6. Continue with catheter insertion to appropriate distance per manufacturer's instructions.

* If you have any questions or would like additional reference information, please contact Arrow International, Inc.

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* Om du har frågor eller önskar ytterligare referensinformation, kontakta Arrow International, Inc.