# Peel-Away Sheath over Dilator Assembly

## Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged. **Warning: Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.**

Do not alter the assembly during insertion, use or removal.

Procedure must be performed by trained personnel well versed in anatomical landmarks, safe techniques, and potential complications.

## Indications for Use:

The Peel-away Sheath over Dilator Assembly permits venous access for catheter introduction.

## Contraindications:

None known.

## Warnings and Precautions:\*

1. **Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.**
2. **Warning: Practitioners must be aware of complications associated with peripheral intravenous access including sheath embolus, catheter-related infection, inadvertent arterial puncture, nerve damage, hematoma, and hemorrhage.**
3. **Warning: Do not leave peel-away sheath in place as an IV access site.**
4. **Warning: Due to the risk of exposure to Human Immunodeficiency Virus (HIV) or other blood borne pathogens, health care professionals should always use universal blood and body fluid precautions in the care of all patients.**

## A Suggested Procedure:

### Use sterile technique.

1. Insert spring-wire guide per manufacturer's instructions. **Precaution: Maintain firm grip on spring-wire guide at all times.**
2. Prepare sheath/dilator assembly by firmly pushing dilator into sheath hub until dilator snaps in place.

3. With spring-wire guide in place, thread tapered tip of sheath/dilator assembly over spring-wire guide. Grasping near skin, advance assembly with slight twisting motion to enlarge subcutaneous track and entry into vein. Insert to a depth where sheath edge is within vessel. Dilator may be partially withdrawn to facilitate advancement of sheath through tortuous vessel. **Precaution: Do not withdraw dilator until sheath edge is within vessel to minimize the risk of damage to sheath tip. Precaution: Sufficient guide wire length must remain exposed at hub end of sheath to maintain a firm grip on guide wire.**
4. Advance peel-away sheath over dilator into vessel, again grasping near skin and using slight twisting motion.
5. To check sheath placement, hold sheath in place and withdraw spring-wire guide and dilator sufficiently to allow venous blood flow.
6. Holding sheath in place, remove guide wire and dilator as a unit. **Warning, to minimize the risk of possible vessel wall perforation do not leave vessel dilator in place as an indwelling catheter. Warning: Although the incidence of spring-wire guide failure is extremely low, practitioner should be aware of potential for breakage if undue force is applied to guide wire.**
7. Using fluoroscopy, if available, advance indwelling catheter through peel-away sheath. If resistance is met while advancing catheter, retract and/or gently flush catheter while advancing.
8. Stop advancing catheter 5 inches (13 cm) before reaching indwelling tip position.
9. Withdraw peel-away sheath until free from venipuncture site.
10. Grasp tabs of peel-away sheath and pull apart, away from indwelling catheter, until sheath splits down entire length.
11. Continue with catheter insertion per manufacturer's instructions.

\* If you have any questions, please contact Arrow International, Inc.

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- \* Om du har frågor, kontakta Arrow International, Inc.