

ARROW[®]

PRESSURE TRANSDUCTION PROBE

The color of the blood aspirated during a central vein cannulation is not always a reliable indicator of venous access. When pressure wave form transduction is desired or indicated this pressure transduction probe may be used in conjunction with the ARROW[®] Raulerson Syringe. This pressure transduction probe can be used with a pressure transduction system as follows:

1. Attach luer fitting of probe to pressure transduction system and flush.
2. Insert probe into the back of the plunger of the ARROW[®] Raulerson Syringe until the nose of hub presses into back of plunger.
3. Observe wave form to confirm venous location of cannula.
4. Remove probe from syringe and discard.