# StatLock® PICC Plus Stabilization Device

## Key Points to StatLock® PICC Plus Stabilization Device Usage:

### Usage:

- PREPARE TARGET AREA WITH SKIN PREP (may or may not be included) and ALLOW TO DRY COMPLETELY.
- Place suture holes in catheter wing over the StatLock® stabilization device posts (prior to pad adherence). Close lids, peel off paper backing, and place on target skin area.
- To remove, lift plastic lids one side at a time.
- REMOVE THE StatLock® STABILIZATION DEVICE ANCHOR PAD ADHESIVE WITH ALCOHOL.

## Read carefully before use.

### Safety and Efficacy Considerations:

The StatLock® stabilization device is for single use only. Rx only. Sterile only when included as a component in a sterile kit. This package does not maintain sterility; use once catheter kit package is opened. Discard if package is opened or damaged.

Do not alter the StatLock® stabilization device or components. Procedure must be performed by trained personnel with knowledge of anatomical landmarks, safe technique, and potential complications.

### Device Description:

StatLock® stabilization device is a stabilization device for compatible winged catheters.

### Contraindications:

Known tape or adhesive allergies.

**Caution:** After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

### Warnings and Cautions

1. Do not use the StatLock® stabilization device where loss of adherence could occur, such as with a confused patient or nonadherent skin.
2. Observe universal blood and body fluid precautions, during application and removal of the StatLock® stabilization device.
3. Avoid the StatLock® stabilization device contact with alcohol or acetone, both can weaken bonding of components and the StatLock® stabilization device pad adherence.

4. Minimize catheter/tube manipulation during application and removal of the StatLock® stabilization device.
5. Always apply adhesive strip to central venous catheters at or near insertion site during dressing change involving the StatLock® stabilization device.
6. Remove oil and moisturizer from targeted skin area.
7. The StatLock® stabilization device should be monitored and replaced in accordance with facility protocol.
8. **Warning:** Intended for single use. Do not reuse. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness, or death of the patient.
9. **Warning:** Do not re-sterilize. The sterility of the single use device is not guaranteed following re-sterilization because of an indeterminable degree of potential pyrogenic or microbial contamination which may lead to infectious complications. Re-sterilization may compromise the structural integrity, essential material and/or design characteristics and may lead to and unpredictable loss of functionality and/or device failure.
10. Federal (USA) law restricts this device to sale by or on the order of a physician.

Use aseptic technique for application and removal of the StatLock® stabilization device.

Do not use if package is damaged

Do not sterilize

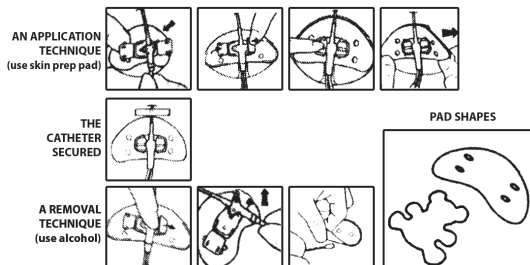
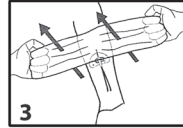
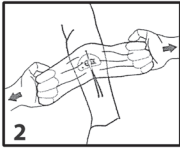
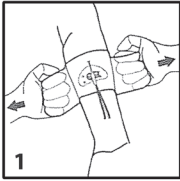
Do not reuse

Lot number

This product and packaging does not contain natural rubber latex or DEHP.

**Important:**

Use illustrated "stretch technique" when removing transparent dressing.

**Note:**

- The StatLock® Stabilization device pad can be adhered after confirmation of proper catheter/tube position.
- Apply adhesive strip to central venous catheters for added security during dressing changes involving the StatLock® stabilization device.

*Assembled in Mexico.*

*Bard and StatLock® are trademarks and/or registered trademarks of C. R. Bard, Inc.*

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*Bard y StatLock son marcas comerciales o marcas comerciales registradas de C. R. Bard, Inc.*
**Bard Access Systems, Inc.**
**Arrow International LLC**