en You-Bend™ Two-Lumen Hemodialysis Catheterization Set with Blue FlexTip® ARROWgard Blue® Catheter for High Volume Infusions

Contents:

- 1: Two-Lumen Indwelling Catheter*: 12 Fr. x 16 cm Radiopaque Polyurethane with Blue FlexTip®, ARROWgard® Antimicrobial Surface Treatment, Extension Line Clamps
- 1: Spring-Wire Guide, Marked: .035" (0.89 mm) dia. x 23-5/8" (60 cm) (Straight Soft Tip on One End - "J" Tip on Other) with Arrow Advancer
- 1: Catheter: 18 Ga. x 2-1/2" (6.35 cm) Radiopaque over 20 Ga. RW Introducer Needle
- 1: Introducer Needle: 18 Ga. x 2-1/2" (6.35 cm) XTW
- 1: Pressure Transduction Probe
- 1: Arrow® Raulerson Spring-Wire Introduction Syringe: 5 ml
- 1: Tissue Dilator: 12 Fr. (4.0 mm) x 14 cm
- 2: Dust Cap: Non-Vented
- 1: Drape: 24" x 36" (61 cm x 91 cm)
- 1: Scalpel: #11

All components are CE 0086 unless otherwise noted.

Not made with natural rubber latex.

* MR Conditional

Fluid path components are non-pyrogenic.

Contraindications: The Arrow® large-bore two-lumen catheter is not designed for long-term hemodialysis or for use in patients with thrombosed vessels. The ARROWgard Blue® antimicrobial surface catheter is contraindicated for patients with known hypersensitivity to chlorhexidine, silver sulfadiazine, and/or sulfa drugs (refer to product instructions for references).

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