BD Eclipse™ Needle

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



Arrow International LLC
Subsidiary of Teleflex Incorporated

3015 Carrington Mill Blvd., Morrisville, NC 27560 USA
USA: 1 866 246 6990 | International: +1 919 544 8000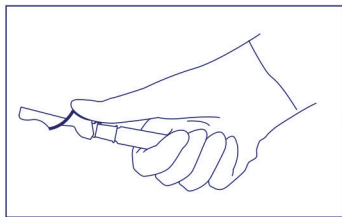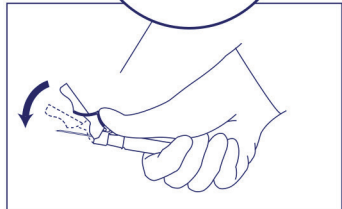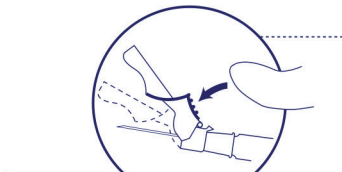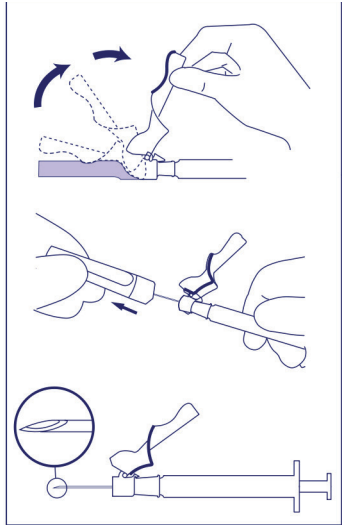

en Attach the BD Eclipse™ Needle to a BD Luer-Lok™ Syringe. Pull back on the safety cover and remove the needle shield.

Bevel Up = Safety Cover Up

Draw up and administer medication in accordance with established protocol.

Use one handed technique and activate away from self and others. For greatest safety, ONLY use the wide textured finger pad area to activate the safety cover.

Activation of the protective mechanism may cause minimal splatter of any fluid that is remaining on the needle after injection.

Discard after single use in an approved sharps container in accordance with applicable regulations and institutional policy.

Non-pyrogenic. Do not use if individual packaging is damaged.

Not made with natural rubber latex.

Do Not Reuse. ~~☒~~

CAUTION

The BD Eclipse™ Needle is not to be used with a Luer Slip Syringe as the needle may become disengaged from the syringe when activating the safety cover.

Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration.

USA only: OSHA standards require that such recapping must be accomplished using a one handed technique. DO NOT hold the needle shield during the recapping process. USA only: Caution: Federal (USA) law restricts this device to sale by or on the order of physician.

To help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, activate the protective mechanism immediately after use.

Do not autoclave BD Eclipse™ Needle before use.

Re-use may lead to infection or other illness/injury.

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