# Teleflex®

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**en** Intended Purpose: The Grip-Lok is intended to provide catheter securement.

## Contraindications: None known.

## Clinical Benefits Statement: The Grip-Lok permits securement of catheter to patient.

## Warnings:

1. Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.

2. Read all package insert warnings, precautions and instructions prior to use. Failure to do so may result in severe patient injury or death.

## INSTRUCTIONS

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.

2. **Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.

3. **Open** the top flap, remove the interior liner and hold the flap open.

4. **Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.

5. **Secure** the top flap over the catheter.

6. **Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

**Direction:** for added skin protection, skin prep may be used prior to application of securement device.

**Note:** replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

**Note:** use of an alcohol swab may help removal of securement device from the skin.

**Note:** Hypoallergenic

A pdf copy of this IFU is located at [www.teleflex.com/IFU](http://www.teleflex.com/IFU)

SZ-00400-100B, Rev. 01 (2022-04)

Manufactured by: TIDI™ Products, LLC 570 Enterprise Drive Neenah, WI 54956 USA

*Grip-Lok is a registered trademark of TIDI Products, LLC.*

**Arrow International LLC** Subsidiary of Teleflex Incorporated 3015 Carrington Mill Blvd., Morrisville, NC 27560 USA USA: 1 866 246 6990 | International: +1 919 544 8000

**Teleflex®**

EU Authorized Representative and Importer: Teleflex Medical IDA Business and Technology Park Dublin Road, Athlone, Co. Westmeath, Ireland

CE REP

2797

EC REP

EU Authorized Representative and Importer:

Teleflex Medical IDA Business and Technology Park Dublin Road, Athlone, Co. Westmeath, Ireland

CE

2797

EC REP

EU Authorized Representative and Importer:

Teleflex Medical IDA Business and Technology Park Dublin Road, Athlone, Co. Westmeath, Ireland

CE

2797

EC REP

EU Authorized Representative and Importer:

Teleflex Medical IDA Business and Technology Park Dublin Road, Athlone, Co. Westmeath, Ireland

CE

2797

EC REP

EU Authorized Representative and Importer:

Teleflex Medical IDA Business and Technology Park Dublin Road, Athlone, Co. Westmeath, Ireland

CE

2797

EC REP

EU Authorized Representative and Importer:

Teleflex Medical IDA Business and Technology Park Dublin Road, Athlone, Co. Westmeath, Ireland

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