Teleflex®

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en Intended Purpose:

The Grip-Lok is intended to provide catheter securement.

Contraindications:

None known.

Clinical Benefits Statement:

The Grip-Lok permits securement of catheter to patient.

Warnings:

- Sterile, Single use:** Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.
- Read all package insert warnings, precautions and instructions prior to use. Failure to do so may result in severe patient injury or death.

INSTRUCTIONS

- Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
- Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open** the top flap, remove the interior liner and hold the flap open.
- Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
- Secure** the top flap over the catheter.
- Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: use of an alcohol swab may help removal of securement device from the skin.

Note: Hypoallergenic

A pdf copy of this IFU is located at www.teleflex.com/IFU
SZ-00400-103A, Rev. 01 (2024-10)

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Grip-Lok is a registered trademark of TIDI Products, LLC.