Inspected Dimensions:
Folded width: 5-1/2" (14 cm)
Folded length: 7-1/2" (19 cm)

ARROW®

TwistLock™ Adapter

Safety and Efficacy Considerations:

Do not use if package has been previously opened or damaged. Warning: Prior to use read all package insert warnings, precautions, and instructions. Failure to do so may result in severe patient injury or death.

Warning: Sterile, Single use: Do not reuse, reprocess or resterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death.

Do not alter any kit/set component during insertion, use, or removal.

Instructions for Use:

1. After successful positioning of the catheter, notice the indicator window between the upper and lower halves of the adapter. The indicator stop should be in the open (O) position (refer to Fig. 1).
2. While maintaining the catheter position, twist the upper and lower halves of the adapter in opposite directions (refer to Fig. 1).

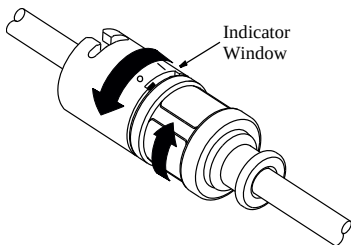
Fig. 1

3. The indicator stop should now show that the catheter is locked (I) in position (refer to Fig. 2).

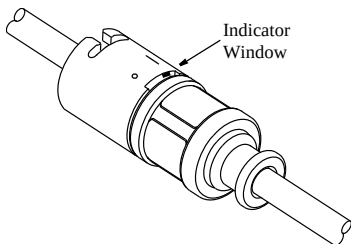
Fig. 2

4. Test the adapter by gently tugging on the catheter to ensure a secure grip on the catheter, and continue the procedure.

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