



You-Bend™ Two-Lumen Hemodialysis Catheterization Set with Blue FlexTip® ARROWg⁺ard Blue® Catheter for High Volume Infusions

REF CU-22122-F

12^{Fr.} | 2^{Lumen} | 16^{cm} catheter length | .035^{inch dia.} spring-wire guide

Contents:

- 1: Two-Lumen Indwelling Catheter†: 12 Fr. x 16 cm Radiopaque Polyurethane with Blue FlexTip®, ARROWg⁺ard® Antimicrobial Surface Treatment¹, Extension Line Clamps
- 1: Spring-Wire Guide, Marked: .035" (0.89 mm) dia. x 23-5/8" (60 cm) (Straight Soft Tip on One End - "J" Tip on Other) with Arrow Advancer
- 1: Catheter: 18 Ga. x 2-1/2" (6.35 cm) Radiopaque over 20 Ga. RW Introducer Needle
- 1: Introducer Needle: 18 Ga. x 2-1/2" (6.35 cm) XTW
- 1: Pressure Transduction Probe
- 1: Arrow® Raulerson Spring-Wire Introduction Syringe: 5 mL
- 1: Tissue Dilator: 12 Fr. (4.0 mm) x 14 cm
- 2: Dust Cap: Non-Vented
- 1: Drape: 24" x 36" (61 cm x 91 cm)
- 1: Scalpel: #11

¹Licensed under US Patent No. 6,706,024.

All components are CE 0086 unless otherwise noted.

Rx only. **Warning: Read all package insert warnings, cautions, and instructions prior to use. Failure to do so may result in severe patient injury or death.**

California Prop. 65
Warning: Cancer and Reproductive Harm
www.P65Warnings.ca.gov.

Fluid path components are non-pyrogenic.

Contraindications: The Arrow large-bore two-lumen catheter is not designed for long-term hemodialysis or for use in patients with thrombosed vessels.

The ARROWg⁺ard Blue antimicrobial surface catheter is contraindicated for patients with known hypersensitivity to chlorhexidine, silver sulfadiazine, and/or sulfa drugs (refer to product instructions for references).



STERILE EO



Not made with natural rubber latex



† MR Conditional



Store below 25°C (77°F).
Avoid excessive heat above 40°C (104°F).

- fr** Jeu de cathétérisme pour hémodialyse à deux lumières You-Bend avec cathéter Blue FlexTip ARROWg⁺ard Blue pour les perfusions de grand volume
- de** Besteck für zweilumigen You-Bend-Hämodialysekatheter mit Blue-FlexTip-ARROWg⁺ard-Blue-Katheter für Infusionen großer Flüssigkeitsmengen
- it** Set per cateterismo in emodialisi a due lumi You-Bend con catetere Blue FlexTip ARROWg⁺ard Blue per volumi di infusione elevati
- pl** Dwukanałowy zestaw You-Bend do hemodializy z cewnikiem Blue FlexTip ARROWg⁺ard Blue do przetwarzania dużych objętości płynów infuzyjnych
- pt** Conjunto de Cateterização para Hemodiálise You-Bend com Lúmen Duplo e Cateter Blue FlexTip, ARROWg⁺ard Blue, para Infusões de Grandes Volumes
- ru** Двухпросветный комплект для гемодиализной катетеризации You-Bend с катетером Blue FlexTip ARROWg⁺ard Blue для вливания больших объемов
- sl** Komplet za hemodializno katetrizacijo You-Bend z dvema svetlinama s katetrom Blue FlexTip ARROWg⁺ard Blue za infundiranje velike prostornine
- es** Conjunto de cateterización de dos luces para hemodiálisis You-Bend con catéter Blue FlexTip ARROWg⁺ard Blue para infusiones de gran volumen
- sv** You-Bend set för tvåkanalig hemodialyskatetrisering med Blue FlexTip ARROWg⁺ard Blue-kateter för infusioner med hög volym
- tr** You-Bend İki Lümenli Hemodiyaliz Kateterizasyon Seti, Blue FlexTip ARROWg⁺ard Blue Kateterli, Yüksek Hacimli İnfüzyonlar için

		Priming Volume*	Flow Rate†	Venous Pressure Setting
	Lumen	(mL)	(mL/min)	(mmHg)
	Distal (12 Ga.)	1.60	220	100
	Proximal (12 Ga.)	1.50	370	200

* Priming volumes are approximate and are done without accessories.
† Blood flow rates are approximate and were determined by varying the venous pressure setting from 100 to 200 mmHg in an in vitro experiment using blood analog solution.

Manufacturer:

Arrow International, Inc.
Subsidiary of Teleflex Incorporated
2400 Bernville Road
Reading, PA 19605 USA

EC REP
Teleflex Medical
IDA Business and Technology Park
Dublin Road, Athlone
Co. Westmeath, Ireland
0086

teleflex®
www.teleflex.com

IU-22122-133A (07/18)



You-Bend™ Two-Lumen Hemodialysis Catheterization Set with Blue FlexTip® ARROWg⁺ard Blue® Catheter for High Volume Infusions

REF CU-22122-F

12^{Fr.} | 2^{Lumen} | 16^{cm} catheter length | .035^{inch dia.} spring-wire guide