



# ARROWgard Blue® You-Bend™ Two-Lumen Hemodialysis Catheterization Kit for High Volume Infusions

REF **AU-25122-F**

**12** Fr. | **2** Lumen | **20** cm catheter length | **.035** inch dia. spring-wire guide

**Contents:**

- |  |  |
|--|--|
| <ul style="list-style-type: none"> <li>1: Two-Lumen Indwelling Catheter†: 12 Fr. x 20 cm Radiopaque Polyurethane with Blue FlexTip®, ARROWgard® Antimicrobial Surface Treatment¹, Extension Line Clamps</li> <li>1: Spring-Wire Guide, Marked: .035" (0.89 mm) dia. x 26-3/4" (68 cm) (Straight Soft Tip on One End - "J" Tip on Other) with Arrow Advancer</li> <li>1: Catheter: 18 Ga. x 2-1/2" (6.35 cm) Radiopaque over 20 Ga. RW Introducer Needle</li> <li>1: Injection Needle: 25 Ga. x 1" (2.54 cm)</li> <li>1: Local Anesthetic Component</li> <li>1: Pressure Transduction Probe</li> <li>1: Introducer Needle: 18 Ga. x 2-1/2" (6.35 cm) XTW and 5 mL Arrow® Raulerson Spring-Wire Introduction Syringe</li> <li>1: Injection Needle: 22 Ga. x 1-1/2" (3.81 cm) and 5 mL Luer-Slip Syringe</li> <li>1: Syringe: 3 mL Luer-Lock</li> </ul> | <ul style="list-style-type: none"> <li>1: Tissue Dilator: 12 Fr. (4.0 mm) x 14 cm</li> <li>1: 5 mL 1% Lidocaine HCl Solution</li> <li>1: 3 mL Applicator 2% CHG and 70% IPA Chloraprep®² One-Step Solution with Hi-Lite Orange™ Tint (non-sterile solution)</li> <li>2: Dust Cap: Non-Vented</li> <li>1: SharpsAway® Disposal Cup</li> <li>1: Drape: 24" x 36" (61 cm x 91 cm) with 4" (10 cm) fenestration</li> <li>1: Scalpel: #11</li> <li>1: Medication Label: 1% Lidocaine</li> <li>2: Gauze Pad: 2" x 2" (5 cm x 5 cm)</li> <li>5: Gauze Pad: 4" x 4" (10 cm x 10 cm)</li> <li>1: Suture: 3-0 Braided Silk with Straight Needle</li> </ul> |
|--|--|

¹Licensed under US Patent No. 6,706,024.  
²A registered trademark of CareFusion or one of its subsidiaries.




Rx only. **Warning: Read all package insert warnings, cautions, and instructions prior to use. Failure to do so may result in severe patient injury or death.**  
Sterilized using ethylene oxide unless otherwise indicated in the contents list.  
⚠ **California Prop. 65**  
**Warning: Cancer and Reproductive Harm [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).**  
Fluid path components are non-pyrogenic.

**Contraindications: The Arrow large-bore two-lumen catheter is not designed for long-term hemodialysis or for use in patients with thrombosed vessels.**  
**The ARROWgard Blue antimicrobial surface catheter is contraindicated for patients with known hypersensitivity to chlorhexidine, silver sulfadiazine, and/or sulfa drugs (refer to product instructions for references).**  
Arrow International provides the enclosed medication label(s) for your convenience. Please ensure that these labels are applied to the correct syringe and corresponding medication.

Lumen	Priming Volume* (mL)	Flow Rate† (mL/min)	Venous Pressure Setting (mmHg)
Distal (12 Ga.)	1.80	200	100
		340	200
Proximal (12 Ga.)	1.70	200	100
		340	200

\* Priming volumes are approximate and are done without accessories.  
† Blood flow rates are approximate and were determined by varying the venous pressure setting from 100 to 200 mmHg in an in vitro experiment using blood analog solution.

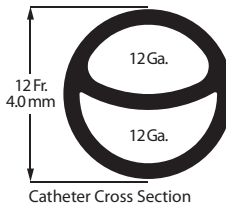

DuPont™ Tyvek®  
packaging

Manufacturer:  
Arrow International LLC  
Subsidiary of Teleflex Incorporated  
3015 Carrington Mill Blvd.  
Morrisville, NC 27560 USA

**Teleflex®**  
[www.teleflex.com](http://www.teleflex.com)

IU-25122-100K, Rev. 01 (2020-12)



# ARROWgard Blue® You-Bend™ Two-Lumen Hemodialysis Catheterization Kit for High Volume Infusions

REF **AU-25122-F**

**12** Fr. | **2** Lumen | **20** cm catheter length | **.035** inch dia. spring-wire guide