



REF **CDC-34052-VPS** (IPNXXXXXX)

5 Fr. **2** Lumen **40** cm catheter length **.018** inch dia. spring-wire guide

ARROW

ErgoPack®

Pressure Injectable Two-Lumen PICC pre-loaded with Arrow® VPS® Stylet



Contents:

- | | |
|---|---|
| 1: Two Lumen TaperFree® Catheter: 5 Fr. (1.81 mm OD) x 40 cm, Pressure Injectable, Blue FlexTip®, Contamination Guard, Arrow® VPS® Stylet and Tuohy-Borst Adapter | 1: Towel |
| 1: GlideThru™ Peel-Away Sheath: 5 Fr. x 2-3/4" (7 cm) Radiopaque over 5 Fr. Dilator | 1: Safety Scalpel: #11 |
| 1: Placement Wire Assembly | 1: Sterile Procedure Sign |
| 1: Spring-Wire Guide, Nitinol, Marked: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on One End - Straight Stiff Tip on Other) with Arrow Advancer | 1: Patient ID Card |
| 1: Injection Needle: SafetyGlide™ ¹ 25 Ga. x 5/8" (1.60 cm) | 1: Chart Sticker |
| 1: Safety Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW | 1: Patient Information Booklet |
| 1: Blunt Fill Needle: 18 Ga. x 1-1/2" (3.81 cm) | 1: Medication Label: 1% Lidocaine |
| 1: Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW | 1: Checklist/CLIP Sheet |
| 1: Syringe: 10 mL Luer-Lock | 2: Paper Tape Measure |
| 1: Syringe: 3 mL Luer-Lock | 1: Tourniquet |
| 4: 10 mL Luer-Lock Pre-Filled Saline Syringe | 1: Foam Electrodes (3 per pack) |
| 1: 5 mL 1% Lidocaine HCl Solution and Alcohol Prep | 10: Gauze Pad: 2" x 2" (5 cm x 5 cm) |
| 1: 3 mL Applicator 2% CHG and 70% IPA ChloraPrep® ² One-Step Solution | 10: Gauze Pad: 4" x 4" (10 cm x 10 cm) |
| 1: 3 mL Applicator 2% CHG and 70% IPA ChloraPrep® ² One-Step Solution with Hi-Lite Orange™ Tint | 1: Surgical Apparel: Impervious Gown |
| 2: Dust Cap: Non-Vented | 1: Dressing: I.V. Advanced 8.5 cm x 11.5 cm |
| 1: SecondSite™ Adjustable Hub: Fastener | 1: Dressing: BIOPATCH® ³ |
| 1: SecondSite™ Adjustable Hub: Catheter Clamp | 1: Surgical Apparel: Mask with Eye Shield |
| 1: SharpsAway® II Locking Disposal Cup | 1: Dressing: STATLOCK® ⁴ PICC Plus Catheter Stabilization Device |
| 1: SharpsAway® Disposal Cup | 1: Surgical Apparel: Bouffant Cap |
| 1: Catheter Trimmer | 1: Surgical Apparel: Mask |
| 1: Towel: 24" x 36" (61 cm x 91 cm) | 1: Tape: Steri-Strip® ⁵ |
| 1: Maximal Barrier Drape™ with two 13 cm (5") x 15 cm (6") fenestrations | 1: HemoHopper® Fluid Receptacle |

Not made with natural rubber latex.

Store between 20 – 25°C (68 – 77°F).

Arrow International provides the enclosed medication label(s) for your convenience. Please ensure that these labels are applied to the correct syringe and corresponding medication.

Contraindications: The Pressure Injectable PICC is contraindicated wherever there is presence of device related infections or presence of thrombosis in the intended insertion vessel or catheter pathway. Clinical assessment of the patient must be completed to ensure no contraindications exist.



Lumen

	Priming Volume* (mL)	Gravity Flow Rate† (mL/hr)	Pump Flow Rate†† (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Distal (18 Ga.)	0.44	568	4720	5
Proximal (18 Ga.)	0.44	568	4680	5

* Priming volumes are approximate and are done without accessories.

† Flow rate values are approximate and are determined using deionized water at 100 cm head height.

†† Pump flow rates are determined at maximum pump pressure of 10 psig and represent approximate flow capabilities.

** Pressure injection flow rates are determined at the injector pressure setting of 300 psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing.

