



REF AK-22122-CDC (IPNXXXXXX)

12 Fr. 2 Lumen 16 cm catheter length .035 inch dia. spring-wire guide

Arrowg+ard Blue® Two-Lumen Hemodialysis Catheterization for High Volume Infusions

Contents:

- 1: Two-Lumen Catheter: 12 Fr. (4.06 mm OD) x 16 cm Radiopaque Polyurethane with Blue FlexTip®, Arrowg+ard® Antimicrobial Surface Treatment, Extension Line Clamps Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Spring-Wire Guide, Marked: .035" (0.89 mm) dia. x 23-5/8" (60 cm) (Straight Soft Tip on One End - "J" Tip on Other) with Arrow Advancer Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Catheter: 18 Ga. x 2-1/2" (6.35 cm) Radiopaque over 20 Ga. RW Introducer Needle Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Injection Needle: 25 Ga. x 1" (2.54 cm) Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Blunt Fill Needle: 18 Ga. x 1-1/2" (3.81 cm) Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Pressure Transduction Probe Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Introducer Needle: 18 Ga. x 2-1/2" (6.35 cm) XTW and 5 mL Arrow® Raulerson Spring-Wire Introduction Syringe Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Injection Needle: 22 Ga. x 1-1/2" (3.81 cm) and 5 mL Luer-Slip Syringe Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Syringe: 3 mL Luer-Lock Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Tissue Dilator: 12 Fr. (4.0 mm) x 14 cm Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: 5 mL 1% Lidocaine HCl Solution and Alcohol Prep Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: 3 mL Applicator 2% CHG and 70% IPA ChloraPrep®1 One-Step Solution with Hi-Lite Orange™ Tint Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 2: Dust Cap: Non-Vented Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: SharpsAway® II Locking Disposal Cup Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: SharpsAway® Disposal Cup Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Drape: 41" x 55" with 4" fenestration Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Needle Holder Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Safety Scalpel: #11 Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 2: Gauze Pad: 2" x 2" (5 cm x 5 cm) Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 5: Gauze Pad: 4" x 4" (10 cm x 10 cm) Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX
- 1: Suture: 3-0 Silk with Curved Needle Origin: XX XXXX XX XXX XXXXXXXX XXXXXXXX

<sup>1</sup>A registered trademark of CareFusion or one of its subsidiaries.

**Rx only**  
**Warning:** Read all package insert warnings, precautions, and instructions prior to use. Failure to do so may result in severe patient injury or death. [www.teleflex.com/IFU](http://www.teleflex.com/IFU)  
**California Prop. 65**  
⚠ **WARNING:** Cancer and Reproductive Harm.  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)  
**Not made with natural rubber latex.**  
Store between 20 – 25°C (68 – 77°F).  
Sterilized using ethylene oxide unless otherwise indicated in the contents list.  
Fluid path components are non-pyrogenic.  
**Contraindications:** The Arrow large-bore two-lumen catheter is not designed for long-term (≥ 30 days) hemodialysis or for use in patients with thrombosed vessels.  
The Arrowg+ard Blue antimicrobial catheter is contraindicated for patients with known hypersensitivity to chlorhexidine, silver sulfadiazine, and/or sulfa drugs.

Lumen	Priming Volume* (mL)	Flow Rate (mL/min)	Venous Pressure Setting†† (mmHg)
Distal (12 Ga.)	1.4	220 370	100 200
Proximal (12 Ga.)	1.3	220 370	100 200

\* Priming volumes are approximate and are done without accessories.  
†† Blood flow rates are approximate and were determined by varying the venous pressure setting from 100 to 200 mmHg in an in vitro experiment using blood analog solution.



LOT Lot Number

YYYY-MM-DD

UDI

YYYY-MM-DD

(01)XXXXXXXXXXXXX  
(17)YYMMDD  
(11)YYMMDD  
(10)Lot Number



Set Product of xxx  
Packaged in xxx

Arrow International LLC  
Subsidiary of Teleflex Incorporated  
3015 Carrington Mill Blvd.  
Morrisville, NC 27560 USA

Teleflex®  
[www.teleflex.com](http://www.teleflex.com)  
LBL057080 R02 (2025-04)



REF AK-22122-CDC

12 Fr. 2 Lumen 16 cm catheter length .035 inch dia. spring-wire guide



Arrowg+ard Blue® Two-Lumen Hemodialysis Catheterization for High Volume Infusions