





(IPNXXXXXX)

55 cm catheter length

Pressure Injectable Two-Lumen PICC with 130 cm Wire



Contents:

- Two Lumen TaperFree® Catheter: 5 Fr. (1.81 mm OD) x 55 cm, Pressure
- Injectable, Blue FlexTip® and Contamination Guard
- GlideThru™ Peel-Away Sheath: 5 Fr. x 4" (10 cm) Radiopaque over 5 Fr.
- Spring-Wire Guide, Nitinol, Marked: .018" (0.46 mm) dia. x 17-3/4" (45 cm) (Straight Soft Tip on One End - Straight Stiff Tip on Other)
- Guidewire, Marked: .018" (0.46 mm) dia. x 130 cm Hydrophilic-Coated Nitinol with Soft Tip Tungsten Coil
- Introducer Needle: Echogenic 21 Ga. x 2-3/4" (7 cm) TW
- Syringe: 10 mL Luer-Lock
- Dust Cap: Non-Vented
- SecondSite™ Adjustable Hub: Fastener
- SecondSite™ Adjustable Hub: Catheter Clamp
- Catheter Trimmer
- Safety Scalpel: #11
- Patient ID Card
- Patient Information Booklet
- Checklist/CLIP Sheet
- Paper Tape Measure
- Dressing: STATLOCK®1 PICC Plus Catheter Stabilization Device
- ¹A registered trademark of C. R. Bard, Inc.

All components are CE 2797 unless otherwise noted.

Contraindications: The Pressure Injectable PICC is contraindicated wherever there is presence of device related infections or presence of thrombosis in the intended insertion vessel or catheter pathway. Clinical assessment of the patient must be completed to ensure no contraindications exist.

Lumen	Priming Volume* (mL)	Gravity Flow Rate† (mL/hr)	Pump Flow Rate†† (mL/hr)	MAX Pressure Injection Flow Rate** (mL/sec)
Distal (18 Ga.)	0.52	416	3660	5
Proximal (18 Ga.)	0.53	409	3560	5

^{*} Priming volumes are approximate and are done without accessories.





LBL068188 R01 (2025-02)

[†] Flow rate values are approximate and are determined using deionized water at 100 cm head

^{††} Pump flow rates are determined at maximum pump pressure of 10 psig and represent approximate flow capabilities.

^{**} Pressure injection flow rates are determined at the injector pressure setting of 300 psi maximum using media of 11.8 centipoise viscosity, with 152 cm pressure tubing.